swiss made

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Case Report

Splinted Restoration On Two-Piece Zirconia Implants

Z-Systems Z5c, Tissue Level Implant

Dr. Jean Louis ROCHE
Implantologie Dentaire
Saint Laurent du Var - France
www.dr-roche.fr

The lower left first and second molars of a 54 years old female patient were replaced with two new two-piece ceramic implants and restored with splinted crowns. The implants (diameter: 4 mm, length: 10 mm) were placed 12 weeks after extraction (Nov. 23, 2012) with a classical flap elevation procedure without adjunct bone augmentation. Figure 1 and 2 show panoramic x-rays before and after implant surgery.

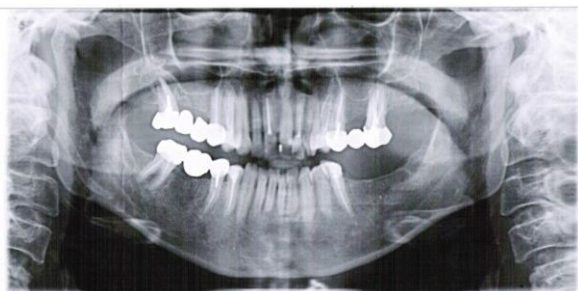
Fig. 1: panoramic x-ray before surgery

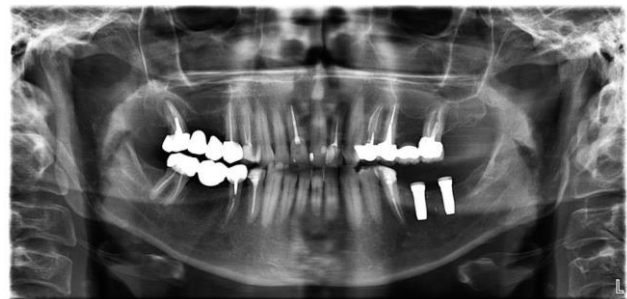
Fig. 2: panoramic x-ray after surgery

The clinical situation after primary wound closure can be seen in figure 3.

Fig. 3: clinical evaluation after suturing

The implants were allowed submucosal healing for 16 weeks (Mar. 3, 2013) before they were restored with straight abutments and splinted crowns in a second stage surgery (see figure 4).

The soft tissue's response around the zirconia implants is great, without any signs of inflammation. The height of the abutments had to be reduced (see figure 5) to let enough space for the future prosthetic restoration.

Fig. 4: clinical evaluation 16 weeks after surgery

Fig. 5: abutment placement

After impression taking the patient was restored with two splinted ceramic crowns. Ten months later (Jan. 10, 2014), the patient was seen for a follow-up examination. Healing was uneventful and gingival health as well as crestal bone level maintenance are very good (see figures 6 and 7).

Fig. 6: clinical evaluation with final restoration in place

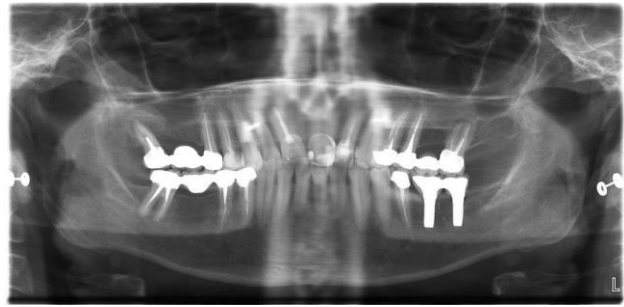
Fig. 7: x-ray follow-up after 10 months

Conclusion:

The new two-piece ceramic implant from Z-Systems shows excellent clinical results and crestal bone level maintenance. Z5c Tissue Level Implant is a reliable option for the prosthetic rehabilitation of missing molars.

Case Report

Delayed placement of two-piece zirconia implants in two patients

Z-Systems Z5c Tissue Level Implant

Noriaki Honma D.D.S P.H.D

Honma Dental Clinic Group Chiba JAPAN

This report is about two first molar replacements in two patients. A female patient of 43 years and a male patient of 46 years, each received a Z5c two-piece zirconia implant (diameter: 4 mm, length: 10 mm) in region 46 (FDI) and were restored with single crowns.

Figures 1&2 show the panoramic x-rays after implant placement.

Fig. 1: panoramic x-ray after implant placement, case1

Fig. 2: panoramic x-ray after implant placement, case2

In both cases the implants were placed after flap elevation and without bone grafting. After submucosal healing of 4-5 months the implants were restored with straight abutments and single crowns. Figures 3&4 show the x-rays after prosthetic restoration

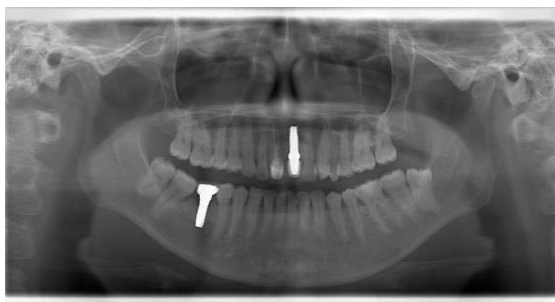
Fig. 3: x.-ray after loading, case 1

Fig. 4: x.-ray after loading, case 2

Meanwhile, both patients could be followed-up for more than one year after final prosthetic rehabilitation. Healing was uneventful, and patients report excellent gingival health and crestal bone level maintenance (see figure5).

Fig. 5: 12 months post loading x-ray follow-up, case1

Conclusion:

As demonstrated in these two cases, the new two-piece zirconia implant from Z-Systems proves to be a reliable alternative to established implant systems.